



***The All Terrain Wheelchair***

## ***STEERING CABLE ADJUSTMENT***

It is important to make sure the steering cables are always tight to give good control, especially when travelling downhill.

Check the steering cables regularly by squeezing them inwards at the rear fork. If you are able to squeeze them inwards a lot then they need tightening.

To tighten cables:

- Ensure steering lever and rear wheel are both straight
- Using a 4mm allen key, loosen one of the cable clamp bolts and pull the cable tight with pliers.
- Then retighten the bolt
- Repeat with the other cable.

It can help to have a friend hold the steering lever straight while you tighten the cables

